
# **Double Cover System**

顱骨圓型接骨板·釘系統

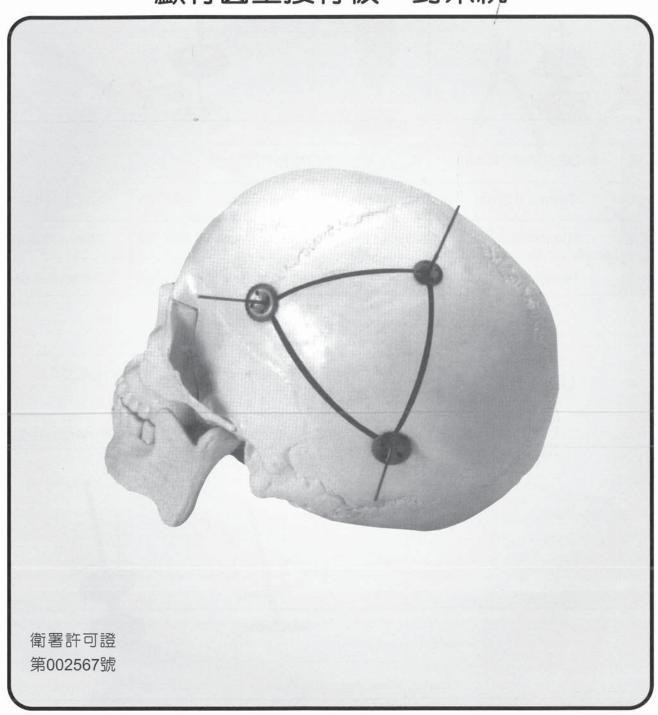



### 亞太醫療器材科技股份有限公司

彰化縣伸港鄉全興工業區工三路2號

電話: (04)798-7099 傳真: (04)798-7077 台北辨事處:

台北市大安區忠孝東路三段96號3樓 電話: (02)2778-7396


電話: (02)2778-7396 傳真: (02)2778-6079

## SYNTEC SCIENTIFIC CORPORATION

2, Kung San Road, Chuan Shing Industrial Zone, Shen Kang, Chang Hua, Taiwan. Tel.:886-4-798-7099 Fax: 886-4-798-7077 TAIPEI OFFICE:


3Fl, No.96, Sec.3, Chung Hsiao East Road, Taipei, Taiwan.

Tel: 886-2-2778-7396 Fax: 886-2-2778-6079

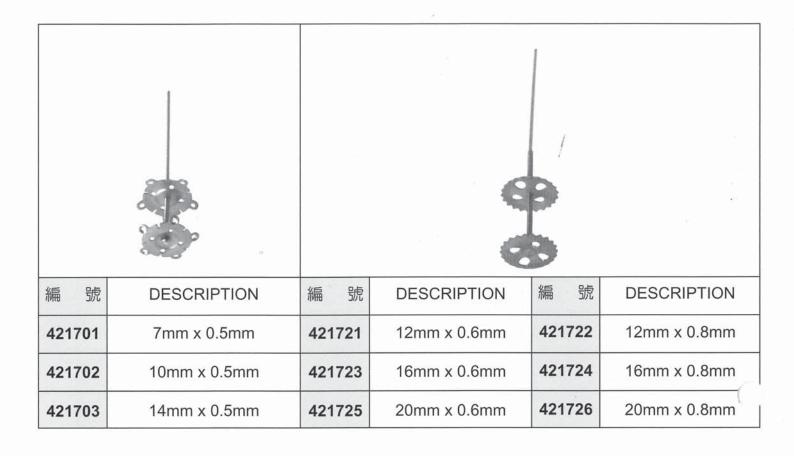

下片傾斜活動設計,適合各部位顱骨的厚度差距,固定更可靠。適用於顱骨骨折的内固定。


The inclined flexible design of the lower plate coan well match the thickness variance of the skull of different area.

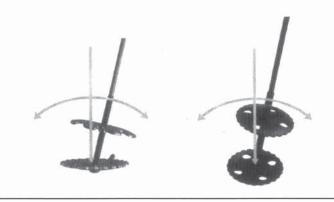


#### 手術操作步驟 Operation Procedure:



Put it around defect area by same interval and bring the plate to the top of guide screw.

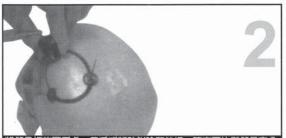

將顧骨板放回原處,用手或輔助鉗玲緊絲桿,確定下片與顧骨充分接觸,順時針旋轉旋緊器,將上接骨片向下移至顱骨表面旋緊。 Replace the skull table and screw down the screw down the guide screw by hand or an assistant pliers. Make sure that the lower plate contacts the skull completely and turn the tightener clockwise to fix the upper plates.

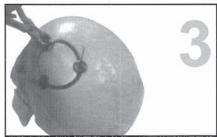

當確定所有鎖片鎖緊和定位後,沈頭前斷鉗逐個剪 斷。

After having made sure that all locking plates have been licked and fixed, sheat off the guide screw with a sinker cropper.

工具 Instruments







DC-02 沈實剪斷鉗 Countersunk headsnip pliers



DC-03 輔助鉗 Assistant Pliers





# **Quality Assurance System Certified**

- USA/FDA 510K Certified
- **CE MARK/MDD Certified**
- GMP Certified
- ISO 9001, 13485 Certified



# SYNTEC SCIENTIFIC CORPORATION

#### FACTORY:

Shen Kang, Chang Hua, Taiwan.

Tel: 886-4-798-7099 Fax: 886-4-798-7077

### TAIPEI OFFICE:

2, Kung San Road, Chuan Shing Industrial Zone, 3Fl., No.96, Sec.3, Chung Hsiao East Road, Taipei, Taiwan.

Tel: 886-2-2778-7396 Fax: 886-2-2778-6079